



For moistening and decontamination of nasal vestibules by physical cleansing

octenisan® md nasal gel

Our plus:

- decontamination by physical cleaning of the nasal vestibules
- to support wound treatment of lesions in the nasal epithelium
- for moistening the nasal vestibule area

Application areas


octenisan® md Nasal Gel is used for moistening and decontamination by physical cleaning of the nasal vestibules. External influences can cause the nose to rapidly dry out and become irritated. This can be remedied by supplying the nasal vestibules with sufficient moisture. The moisturising effects of hydrogel promote a favorable physiological environment which helps to prevent the settlement of bacteria. Restoring the functions of protection and cleaning contribute to effective decontamination of the nasal vestibules. Lesions of the nasal epithelium are also supplied with moisture through the hydrogel and thus can heal optimally.

Instructions for use

Use octenisan® md Nasal Gel up to twice a day as needed. octenisan® md Nasal Gel is for use in the nasal vestibules only. Apply the gel with a cotton bud in a sufficient quantity to the frontal area of the nose (nasal vestibules), so that the surface is completely covered.

Close the nose by pinching and massage between thumb and finger to ensure an even distribution of the gel. If needed, the nose should be thoroughly cleaned beforehand with a wet cotton bud. Excess gel is removed with a cotton bud or swab ensuring that a moist film remains.

Direct contact with the opening of the tube must be avoided to prevent contamination with germs.

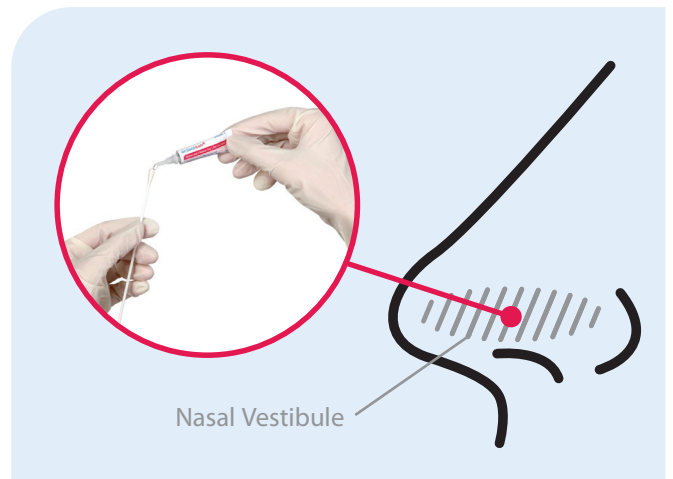

octenisan[®] md nasal gel

Product data

Ingredients:

Purified water, propylene glycol, hydroxyethylcellulose, octenidine HCl

Special advice

octenisan[®] md Nasal Gel must not be used with known or suspected allergy to one of the ingredients (propylene glycol, hydroxyethyl cellulose, octenidine HCl). Consult with your physician, if you have any doubts.

Not suitable for use in children under 1.

Do not use in combination with cleansing soaps, ointments, oils, enzymes or similar, as this may affect the preservation.

Do not use in combination with PVP iodine, as this can lead to discoloration and impaired antiseptic effect of the PVP iodine.

Do not use the octenisan[®] md Nasal Gel without medical supervision for longer than 2 weeks continuously. octenisan[®] md nasal gel must not be used on cartilage, in the eyes or in the ear to avoid possible tissue damage. Do not insert too deep in the nose. Do not use for infusion or injection. Do not swallow.

Do not use if tube is damaged. Store the tube at room temperature protected from direct sunlight.

ARTG 277 976

Information for order

Item	Delivery form	Item no.
octenisan md nasal gel 6 ml	20/Carton	70000690

Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining our high quality standards.