



Rinse agent for automatic reprocessing of medical devices ph-neutral

thermosept® BSK

Our Plus

- ensures a fast and spot-free drying
- optimal surface wetting
- good material compatibility due to neutral pH-value
- Rinse agent for the use in WD, bed- and container washers

Application areas

thermosept® BSK supports rapid, spot-free drying after the automated cleaning and disinfection of surgical instruments, containers also made from anodized aluminium and bedframes. Also suitable for the use within a chemo-thermal reprocessing in bed- and containerwashers. Compatible with chemo- thermal disinfectant component thermosept® NDR.

Instructions for use

Dosage for chemo-thermal reprocessing in bed- and container-washers:

0.1 - 0.2 %, (1 - 2 ml/l water).

Dosage as rinse aid in WD for thermal reprocessing of medical devices:

0.01 - 0.03 %, (0,1 - 0,3 ml/l water).

For the use of thermosept BSK within a chemo-thermal processes the use of thermosept NDR as cleaning/disinfection component is recommended.



Product data

Composition:

15 - 30 % non-ionic surfactants, solubilising agents, corrosion inhibitors, pH-regulator.

Chemical-physical data

Color	colourless
Density	ca. 0,99 g/cm ³ / 20 °C
Flash point	37 °C / Method : DIN 51755 Part 1
Form	liquid
pH	ca. 7 / 20 °C
Viscosity, dynamic	No data available

Special advice

Please consider reprocessing recommendations of machine and instrument manufacturers.

Information for order

Item	Delivery form	Item no.
thermosept BSK -D- 200 kg FA	1/drum(s)	on request
thermosept BSK Drumt. 200 l FA	1/drum(s)	on request
thermosept BSK 5 kg KA	1/Canister	on request
thermosept BSK -D- 20 l KA	1/Canister	on request

These products are not available in every country. For more information please contact our local subsidiary or distributor.

Application aids

Application aids	Item no.
Can key for 20 + 25 l	135901
Can key for 5 + 10 l	135810

Related Products

- thermosept® NDR

Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining out high quality standards.

Expert opinion and information

Please visit our website for an overview of all available literature/reports on the product: www.schuelke.com. For individual questions: Customer Sales Service
Phone: +49 40 52100-666
E-Mail: info@schuelke.com



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.

schülke Headquarters
Schülke & Mayr GmbH
Robert-Koch-Str. 2
22851 Norderstedt, Germany
Phone +49 40 - 52100 - 0
Fax +49 40 - 52100 - 318
www.schuelke.com
info@schuelke.com